BRASSELER **VE**TM

VISION ENHANCEMENT

Loupes User's Manual

BRASSELER USA[®]
DENTAL INSTRUMENTATION

Getting started with Brassler USA[®] Telescopes

Although your new telescopes have been fully tested and adjusted by our factory technicians, they may need to be initially customized to your facial anatomy. Please read the following guide to learn how to adjust and care for your new telescopes — now and in the future. If you need help, call a trained technician. See the back of this manual for contact information.

Table of Contents

Before Using – Adjust Your Telescopes	2
Accessories	5
Using Your Telescopes	6
Cleaning Instructions	6
Refurbishing Service	7
Telescope Prescription Service	7
SPANISH INSTRUCTIONS	10
FRENCH INSTRUCTIONS	22

Before Using – Adjust Your Telescopes

(Fig. 1)

1. *Adjust Headstrap*

Use of headstrap distributes weight around head, allowing all-day wearing comfort.

- a. Loosen bead.
- b. Place frames on head, tighten bead comfortably.
- c. To remove, loosen bead, grasp temples, pull up and forward (Fig. 1).

(Fig. 2)

2. *Adjust Nosepad*

Squeeze or spread nose pad so that pads sit flat on nose (Fig. 2).

- a. To lower frame, spread pad apart.
- b. To raise frame, squeeze pad together.


3. *Check Working Distance*

Sit in your working position and view the operating site. If your working distance is correct, the image should be in focus. If the image is blurry, you may require an adjustment to your working distance.

- a. If the image comes into focus when you lean in, toward the operating site, then you should consider a **shorter** working distance.
- b. If the image comes into focus as you lean back, further from the operating site, then you should consider a **longer** working distance.

If you need a different working distance, contact your local Brasseler USA representative, or call 1-800-841-4522.

4a. Check Telescope Convergence - Through-the-Lens Loupes

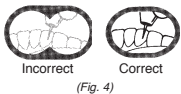(Fig. 4)

(Fig. 5) Raise or lower to gain clear field-of-view


When the focus is properly adjusted, the two circles that form the field-of-view should form an ellipse (Fig. 4).

If the field is not clear, there is double vision, eye strain or no field-of-view:


- a. Reposition frame and/or telescope on your face until there is a clear field (Fig. 5).

Note if you have raised or lowered the frame on your face.

- b. If the frame needs to be lowered – spread the nosepads.
- c. If the frame needs to be raised – squeeze the nosepads together.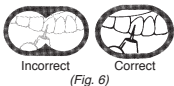


(Fig. 6)

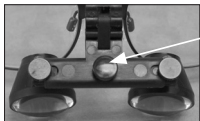(Fig. 7)

4b. Check Telescope Convergence – Flip-Up Loupes

- a. Flip telescopes down so they touch the eyeglass lens, and verify that image is correct (Fig. 6).
- b. If incorrect, rotate pupil knob until you see one oval or circular field without overlapping circles (Fig. 7).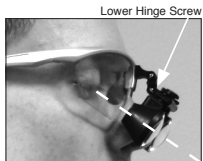

(Fig. 8)

5. Check Downward Angle – Flip-Up Telescopes

Hinge Angle

A comfortable, downward angle has been preset at the factory.

1. In your operating position, with telescopes flipped down, have someone look at them from the side. They should be able to draw an imaginary line through the middle of the telescope to the operating site.
2. If not, loosen the lower hinge screw and adjust height and angle to match the imaginary line (Fig. 8).

Accessories


1. Side Shields

Your loupe is equipped with side shields to protect your eyes from backsplash. To attach side shields, simply push the two tabs on the top of each shield into the notches on the bottom of each temple arm.

2. Lens Cap Covers

Brasseler USA provides lens cap covers for your loupes. These covers are designed to protect the lens from scratches or damage during air abrasion procedures.

3. Flip Grip (only included with flip-up loupes)

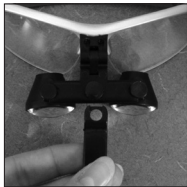The flip grips included with your flip-up loupes allow you to lift the loupes from your field of vision without contaminating your gloves or instruments.

Rinse under fast-running water to remove gross debris. Disinfect using either a quaternary ammonium disinfectant such as CaviCide®, or a phenol alcohol-based spray such as Lysol® brand disinfectant. Follow the label directions.*

*CaviCide and Lysol are not registered trademarks of Brasseler USA

Using Your Telescopes

It is normal to experience some disorientation when first using magnification. Some professionals begin to use the telescopes full-time and some require a breaking-in period.

If you experience difficulty, gradually increase the length of daily use. Generally a break-in period lasts no more than two to three weeks. Call Technical Support if you need assistance.

Cleaning Instructions

Your telescopes are manufactured from the highest quality components and will provide you with a long, useful life, with proper care. These are precision optical instruments that must be properly cleaned and protected from rough use and improper storage. To avoid scratching the telescope lenses, rinse the telescopes and frame under fast-running water to remove gross debris. **DO NOT SUBMERGE IN ANY LIQUID OR AN ULTRASONIC CLEANER OR DAMAGE MAY RESULT.**

You may disinfect using either a quaternary ammonium disinfectant such as CaviCide®, or a phenol alcohol-based spray such as Lysol® brand disinfectant. Follow the label directions. **DO NOT USE AN AUTOCLAVE, CHEMICLAVE, GLUTARALDEHYDE, IODOPHOR OR ANY OTHER STERILIZATION METHOD, OR DAMAGE MAY RESULT.** If there are any questions, please call Customer Service.

Finally, clean all lens surfaces with an optical lens cleaning cloth that is formulated for use with coated lens surfaces. These cleaning cloths can be purchased at major retail chains, optical stores and drug stores.

Telescope Prescription Service

When your eyes change, you can send your telescopes to Brasseler USA to install your new prescription. Package your telescopes and prescription carefully and call Technical Support at 800-841-4522 to make arrangements. Ship them to the address on the back of this manual. **DO NOT SHIP BEFORE CALLING TO MAKE ARRANGEMENTS.**

When coordinating the return of your telescope, please allow sufficient time for the installation of your prescription. If you are sending in a through-the-lens telescope, your new prescription needs to be installed in both the eyeglass lens and inside the telescopes. Please discuss your schedule with Technical Support to coordinate the return shipment of your telescopes.

Warranty, Returns & Exchange Policy

Brasseler USA guarantees our products, their performance and customer satisfaction. If the loupes are not satisfactory to the end user, they can be returned for any reason within 30 days of date of purchase. Items returned showing signs of use, abuse or neglect will not be credited.

Exceptions on Returns/Exchanges

1. Through-the-lens style loupes with prescription added to either loupe frames or back of loupe barrels cannot be returned for any reason other than manufacturing defect.
2. Flip-up style loupe frames and/or loupe barrels with prescription added cannot be returned for money back. Cost of

prescription-added loupe barrels and/or frames cannot be refunded; however cost of an unaltered component (loupe or frame) can be returned for credit if it is found to be in saleable condition and has no visible signs of use. Prescription-added loupe barrels or frames can be combined by Brasseler USA with a new frame or loupe barrel of the customer's choosing.

3. Through-the-lens style loupes with no prescription added to either frames or back of barrels may be exchanged or returned for money back within 30 day trial period as long as product shows no visible signs of use.

Loupes Warranty

Brasseler USA will provide a **limited lifetime warranty** against manufacturing defect on optic barrels and all working parts of Flip-up hinge mechanism. In addition, Brasseler USA will provide a 3-Year limited warranty on all loupe frames, including carrier lenses and side shields.

If frame breakage occurs within the warranty period, Brasseler USA will repair (or replace if they cannot be repaired) loupe frames. Cost of prescription to be added to new frames and cost of re-drilling TTL frames not included in warranty. As with any product, it should be expected that component parts will show wear and tear with use over time and may eventually need to be replaced. Replacement component parts are available. Damage to optic barrels due to customer negligence, misuse or abuse are not covered under the warranty. (Examples of negligence, misuse or abuse would include but are not limited to, submerging the

product in water, wiping the product with chemicals other than those specified in the owner's manual, dropping or mishandling the product or scratching of the lenses).

To return, exchange or obtain warranty repairs, please contact Brasseler USA's Handpiece Inside Account Management Team at 800-841-4522 for a Return Materials Authorization (RMA#) and return label.

When product is returned, the Brasseler USA-assigned RMA# must accompany the product. Brasseler USA is not responsible for any returned product without an assigned RMA#. Return product with RMA# to:

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