


Gold Crown Polishing Kit

David Andrus, C.D.T.

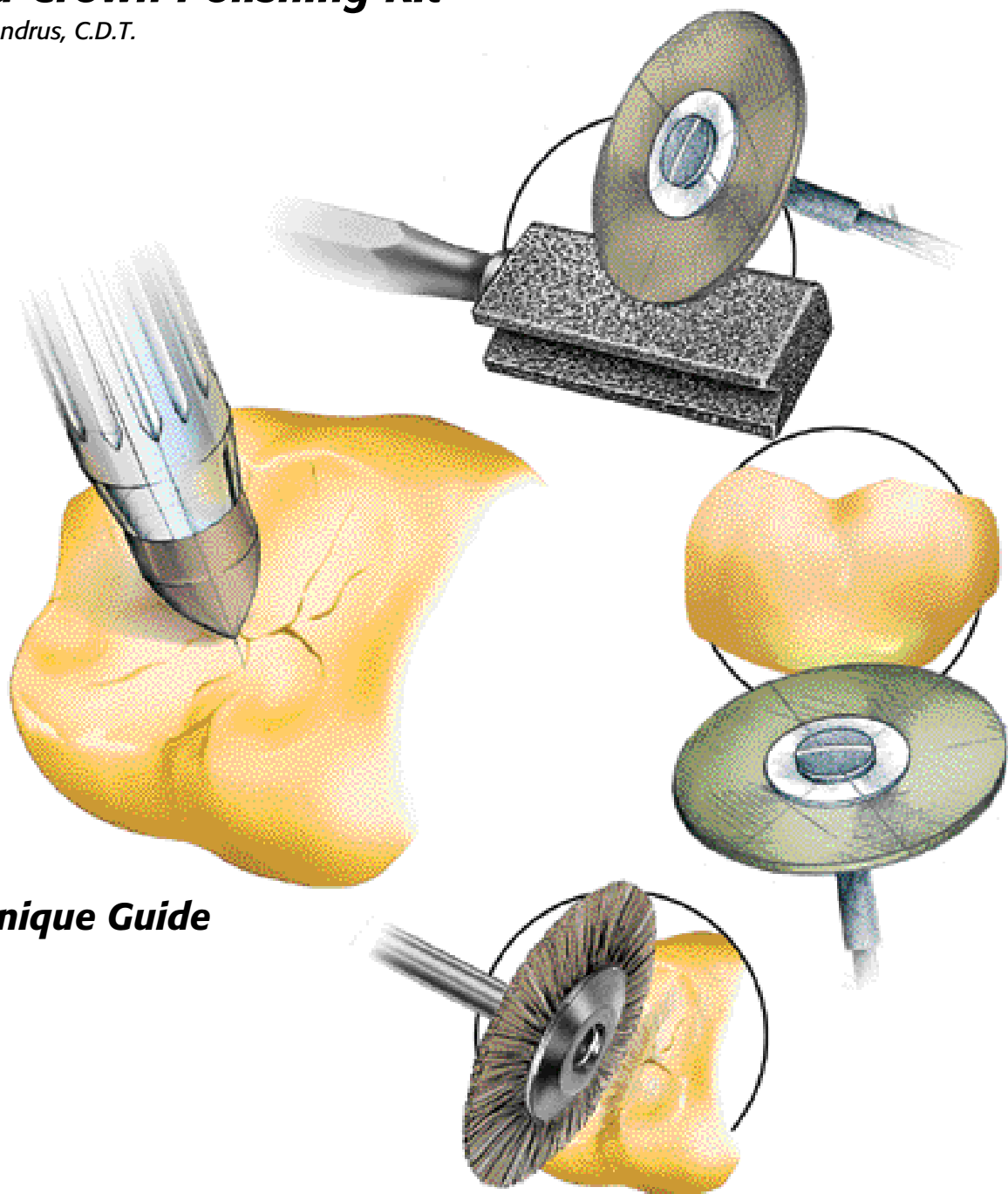

Technique Guide

Consistently. Rationally. Systematically.

Buy Direct. Buy Brasseler USA.



BRASELER USA
DENTAL ROTARY INSTRUMENTS

The Gold Crown Polishing Kit

The Gold Crown Polishing Kit 372.00 by Brasseler USA according to David Andrus, C.D.T. is designed for optimal polishing of cast (gold) or metal restorations. The kit is very simple to use, and when the suggested sequence is followed leads to an ideal polish in an efficient manner. The kit contains three rubber wheels, one large rubber cylinder, one small rubber cylinder, one soft bristle brush, and one rag wheel.

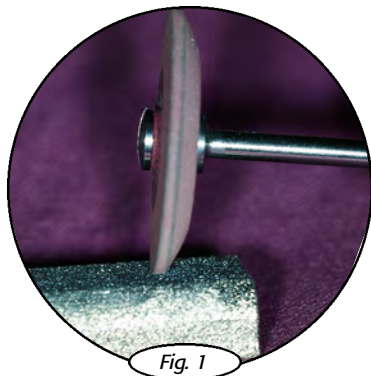


Fig. 1
Contour the 0001-220 brown rubber wheel to a saucer shape with a 16A dressing diamond stone. Working at a 45° angle smooth the contacts and margins to a satiny sheen.

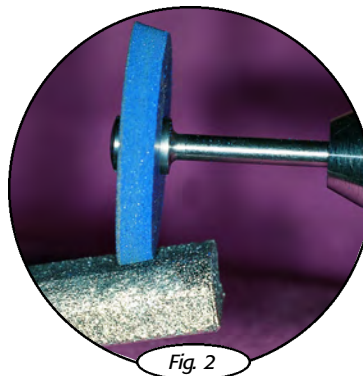


Fig. 2
Contour the edge of the 0401-220 blue rubber wheel to a 45° angle then smooth the buccal, lingual, proximal, and any occlusal surfaces that can be easily reached to a satiny sheen.

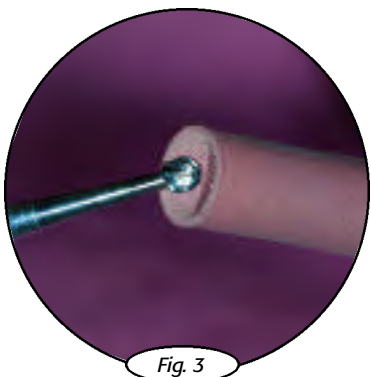


Fig. 3
With the 0023-060 brown rubber cylinder in a handpiece, use a #8 round carbide bur to make a hole in the end and flare it to the shape of a prophy cup. Smooth the occlusal surface (triangular ridges, incline planes, and fossa) to a satiny sheen. The occlusal surface should have a smooth and uniform appearance.

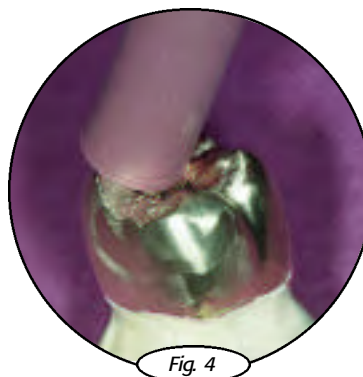


Fig. 5
Using an 1108 dressing stone (not included in kit) shape the 1102-030 red cylinder into a small point and smooth the central pit and deep fossa.


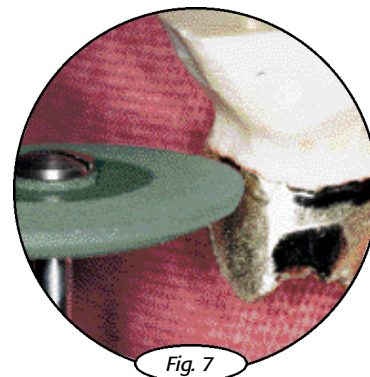


Fig. 7
Contour the 0101-220 green rubber wheel to a knife edge shape and pre-polish the margins, contacts, buccal, lingual, proximal, and any occlusal surfaces that can be reached.



The (0599) #9 Robinson soft bristle brush and a buffing compound is used to buff the entire surface of the crown.

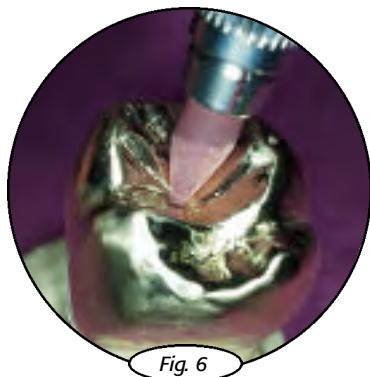


The (0559) #9 Robinson soft bristle brush with rouge is used to polish the crown to a high luster.




A 16422HP muslin high-shine rag wheel with rouge is used to put the final high-shine on the gold crown.

Note: Items not included in kit

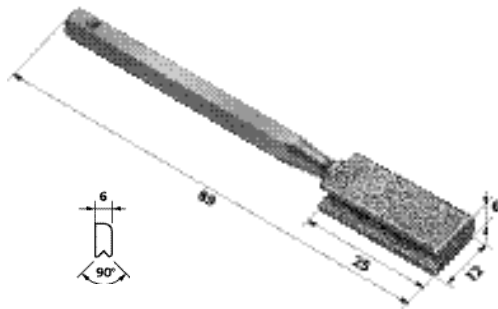

Brasseler USA 16A dressing diamond used to contour polishing wheels.

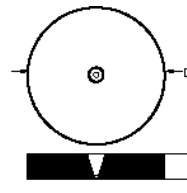


Brasseler USA 1108 dressing stone for contouring rubber points.

Gold Crown Polishing Kit



372.00

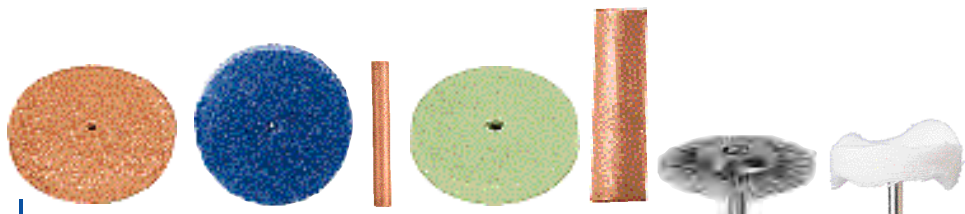

U.S. No.						#9 Soft	
Order No.	0001	0401	1102	0101	0023	0599	16422HP
Size Ø 1/10 mm	220	220	030	220	060	%"	220
L mm	3.0	3.0	22.0	3.0	22.0	-	-

Gold Crown Polishing Kit
Developed by David Andrus C.D.T.

Straight Handpiece 11 – Ø2.35mm x 44.5mm L



Brasseler USA

One Brasseler Blvd.
Savannah, GA 31419

Toll Free: 800.841.4522

Phone: 912.925.8525

Fax: 888.610.1937

Buy Direct. Buy Brasseler USA.

©2003 Brasseler USA




BRASSELER USA
DENTAL ROTARY INSTRUMENTS

B-2122-04.03-KPC

Gold Crown Polishing Kit

David Andrus, C.D.T.

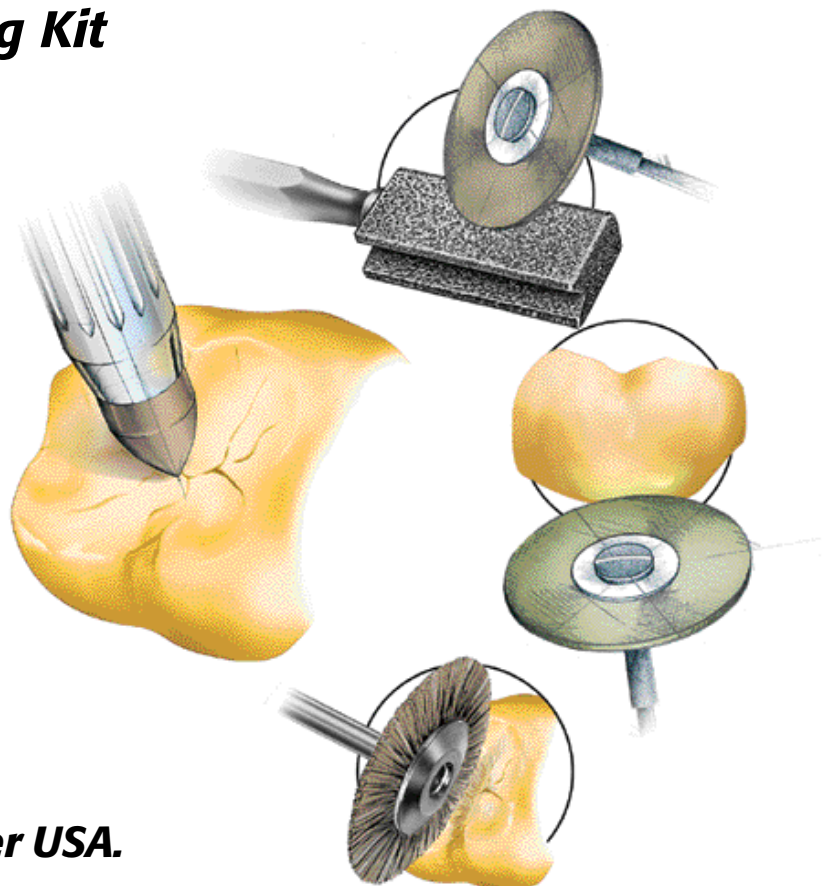

Technique Guide

Buy Direct. Buy Brasseler USA.