


NL45 BASIC HANDPIECE MAINTENANCE



Remove bur from the NL45.

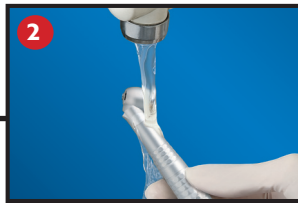

Wash the NL45 under warm water and clean softly with a toothbrush.

Manual

OR

Care3 Plus™

After the proper nozzle for the NL45 is connected, insert into the back end of the NL45 and spray for 2 - 3 seconds or until lubrication is expelled from the head of the handpiece.



Open the door and properly connect the NL45 to the adapter and make certain the bur hole is facing the door. Close the door securely.

Purge the handpiece of excess lubrication either via an air station or by returning to the operatory and placing the handpiece back on its coupler. Purging excess lubricant for up to five seconds is sufficient.



Press "S" mode for the short cleaning cycle. Press the START button and wait until Care3 Plus completes cleaning, lubrication and purging cycle.



PERIODIC MAINTENANCE

Push-button chuck: Approximately once a week it is beneficial to spray the handpiece chuck for 3-5 seconds to insure optimum performance. Depress push button while spraying with PANA spray using pointed nozzle.



Bag and place handpiece in the autoclave. (Remember: Do not autoclave with bur in the chuck)

It is not necessary to spray the handpiece after the autoclave cycle is complete.

Handpiece questions?
Contact Brasseler USA directly!
1.800.841.4522 (ext. 4675)

By Your Side in Dentistry



BRASSLER USA
DENTAL INSTRUMENTATION



To order call 800.841.4522
or fax 888.610.1937.
Visit our website: BrasselerUSA.com

