


---

# FINISHING & POLISHING OF MODERN DENTAL CERAMICS

## Lithium Disilicate

---

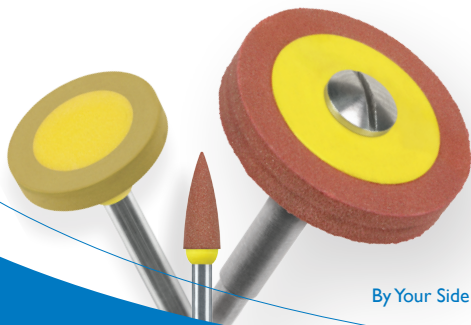

Dialite<sup>®</sup> LD

By Your Side in Dentistry



**BRASSLER USA**  
DENTAL INSTRUMENTATION



Adjustment of occlusion using football-shaped red-band Dialite finishing diamond (8369DF)  
(Recommended Speed: 100K rpm)



Adjusting and contouring with pink medium LD Grinder (LD13M).

## LD Grinders

Tip #1: Heatless adjusting of Lithium Disilicate.



Order #	LD11M	LD13M	LD15M	LD14M	LD11C	LD13C	LD15C	LD14C
Grit	Medium	Medium	Medium	Medium	Coarse	Coarse	Coarse	Coarse
Head Size (1/10 mm)	030	040	150	050	030	040	150	050
Head Length (mm)	7.5	10.0	3.0	13.0	7.5	10.0	3.0	13.0



Dialite LD red medium polishing point (W16MLD) for pre-polishing.



Dialite LD yellow fine polishing point (H2FLD) for high shine.

**Dialite**  
DIAMONDS

Tip #2: Fine adjusting or removal of Zirconia restorations with Dialite Diamonds, available in the K0196 Restoration Removal Kit.



Order #	8801LDF	8369DF	8856DF
Head Size (1/10 mm)	014	025	016
Head Length (mm)	—	5.5	8.0



Lithium disilicate crown polished with Dialite LD.

*Photos and techniques by  
John A. Sorensen, DMD, PhD, University of Washington*

# Dialite<sup>®</sup> LD Finishing and Polishing Systems for Lithium Disilicate



**K0263**  
Dialite<sup>®</sup> LD Intra-Oral Adjustment  
Finishing & Polishing System  
Recommended Speeds: 5,000-7,000 rpms



**K0240**  
Dialite<sup>®</sup> LD Extra-Oral Lithium  
Disilicate Polishing System  
Recommended Speeds: 7,000-10,000 rpms



To order call 800.841.4522 or fax 888.610.1937.  
In Canada call 800.363.3838 or fax 866.330.4454.  
Visit our website at [BrasselerUSA.com](http://BrasselerUSA.com)