

solutions™

DEVELOPED BY BRASSELER USA

DIALITE® INTRA-ORAL PORCELAIN POLISHING SYSTEM



B-3734B-REV2-5022150L0-05.15

©2015 Brasseler USA. All rights reserved. Made in the U.S.A.

Order Number:
K000010



Figure Number	W17DG	W16DG	W18DG	W17DM	W16DM	W18DM
Head Size (1/10 mm)	060	040	100	060	040	100
Head Length (mm)	7.5	10.0	2.5	7.5	10.0	2.5




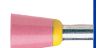







Figure Number	W17D	W16D	W18D
Head Size (1/10 mm)	060	040	100
Head Length (mm)	7.5	10.0	2.5

Only Available Direct from your Brasseler Representative
To order call 800.841.4522 or fax 888.610.1937
In Canada call 800.363.3838 or fax 866.330.4454
Visit our website at BrasselerUSA.com



BRASSELER
USA®

	Order #	Head Size	Grit	Usage
	W16DG-RA	040	C	Polisher - Occlusal Surface
	W17DG-RA	060	C	Polisher - Labial/Lingual/Buccal
	W18DG-RA	100	C	Polisher - Interproximal Surfaces
	W16DM-RA	040	M	Polisher - Occlusal Surface
	W17DM-RA	060	M	Polisher - Labial/Lingual/Buccal

	Order #	Head Size	Grit	Usage
	W18DM-RA	100	M	Polisher - Interproximal Surfaces
	W16D-RA	040	F	Polisher - Occlusal Surface
	W17D-RA	060	F	Polisher - Labial/Lingual/Buccal
	W18D-RA	100	F	Polisher - Interproximal Surfaces

TIPS & TECHNIQUES:

Complimentary items:

Power Suggestion:

Sterilization:

Recommended Speeds:

Polishers produce best results when using all three steps in order (coarse, medium and fine); Dialite is best when used on feldspathic porcelain.

Intra-Oral LD Grinders (LD11M.RA or LD11C.RA) and Dialite Diamonds (Round, Football and Round-End Taper)

Brasseler Forza F1 Electric Attachment with light pressure

Autoclave only

5,000-7,000 rpms



BRASSLER
USA®