

solutions™

LD Grinder System




Order Number:
K0227




Figure Number	LD13C	LD15C	LD14C	LD13M	LD15M	LD14M
Head Size ($\frac{1}{10}$ mm)	040	150	050	040	150	050
Head Length (mm)	10.0	3.0	13.0	10.0	3.0	13.0

Only Available Direct from your Brasser Representative
To order call 800.841.4522 or fax 888.610.1937
In Canada call 800.363.3838 or fax 866.330.4454
Visit our website at BrasserUSA.com

solutions™ K0227 LD Grinder System

	Order #	Head Size	Grit	Usage
	LD15C-HP	150	C	Heatless rinder for ceramics
	LD14C-HP	050	C	Heatless grinder for ceramics
	LD13C-HP	030	C	Heatless grinder for ceramics

	Order #	Head Size	Grit	Usage
	LD15M-HP	150	M	Heatless grinder for ceramics
	LD14M-HP	050	M	Heatless grinder for ceramics
	LD13M-HP	030	M	Heatless grinder for ceramics

TIPS & TECHNIQUES: LD Grinders are ideal for grinding down sprues and heatless grinding on all ceramics: Feldspathic, Lithium Disilicate and Zirconia.

Complimentary items: Dialite ZR and Dialite LD for polishing zirconia or lithium disilicate; 6934.11.250 for cutting off sprues, Dialite Extra-Oral Diamond Paste (5015241U0), Diamond Discs (911H, 911HH, 911HV)

Power Suggestion: Brasseler Forza L50K Laboratory Motor

Sterilization: Autoclave only

Recommended Speeds: 8,000 -12,000 rpms

By Your Side In Dentistry



BRASSLER USA
DENTAL INSTRUMENTATION