


solutions™

Dialite® Adjusting, Finishing
& Polishing System

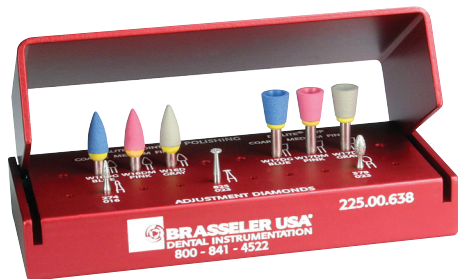

Order Number:
225.00.638



Figure Number	WI6DG	WI6DM	WI6D	WI7DG	WI7DM	WI7D
Head Size ($\frac{1}{10}$ mm)	040	040	040	060	060	060
Head Length (mm)	10.0	10.0	10.0	7.5	7.5	7.5



Figure Number	8801LDF	8369DF	8856DF
Head Size ($\frac{1}{10}$ mm)	014	025	016
Head Length (mm)	—	5.5	8.0





Only Available Direct from your Brasseler Representative
To order call 800.841.4522 or fax 888.610.1937
In Canada call 800.363.3838 or fax 866.330.4454
Visit our website at BrasselerUSA.com






By Your Side In Dentistry



B-3744B-REVI-5022180L0-01.13-RRD

solutions™ 225.00.638 Dialite® Adjusting, Finishing & Polishing System

	Order #	Head Size	Grit	Usage
	W16DCHP-RA	150	C	Polisher - Occlusal Surface
	W16DMHP-RA	050	M	Polisher - Occlusal Surface
	W16DHP-RA	030	F	Polisher - Occlusal Surface
	W17DCHP-RA	150	C	Polisher - Labial/Lingual/Buccal

	Order #	Head Size	Grit	Usage
	W17DMHP-RA	050	M	Polisher - Labial/Lingual/Buccal
	W17DHP-RA	030	F	Polisher - Labial/Lingual/Buccal
	8801LDF-FG	014	F	Diamond - Endo Access/ Small Adjustments
	8369DF-FG	025	F	Diamond - Occlusal Surface
	8856DF-FG	016	F	Diamond - Contact Surfaces/ Crown Removal

TIPS & TECHNIQUES: Polishers produce best results when using all three steps in order (coarse, medium and fine); Use Copious amounts of water when using the diamonds.

Complimentary items: LD Grinders for heatless grinding (LD11C/M)

Power Suggestion: Use a Brasseler Forza FI Electric Attachment at low speed for polishers and Brasseler Forza F5 high speed attachment for diamonds.

Sterilization: Autoclave only

Recommended Speeds: 5,000-7,000 rpms

By Your Side In Dentistry



BRASSLER USA
DENTAL INSTRUMENTATION