

solutions™

Dialite® LD Intra-Oral Polishing System for Lithium Disilicate





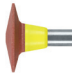
Order Number:
K0241



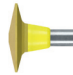


Figure Number	W16MLD.RA	W16FLD.RA	W17MLD.RA	W17FLD.RA	W18MLD.RA	W18FLD.RA
Head Size ($\frac{1}{10}$ mm)	040	040	060	060	100	100
Head Length (mm)	10.0	10.0	7.5	7.5	2.5	2.5

Only Available Direct from your Brasseler Representative
To order call 800.841.4522 or fax 888.610.1937
In Canada call 800.363.3838 or fax 866.330.4454
Visit our website at BrasselerUSA.com

solutions™ K024I Dialite® LD Intra-Oral Polishing System for Lithium Disilicate

	Order #	Head Size	Grit	Usage
	W16MLD.RA	040	M	Polisher - Occlusal Surfaces
	W17MLD.RA	060	M	Polisher - Labial/Lingual/Buccal
	W18MLD.RA	100	M	Polisher - Interproximal Surfaces

	Order #	Head Size	Grit	Usage
	W16FLD.RA	040	F	Polisher - Occlusal Surfaces
	W17FLD.RA	060	F	Polisher - Labial/Lingual/Buccal
	W18FLD.RA	100	F	Polisher - Interproximal Surfaces

TIPS & TECHNIQUES: Polishers produce best results when using both grits in order (medium and fine); medium red for pre-polishing and fine yellow for final polishing. Any adjustments should be made with the LD Grinder in medium pink

Complimentary items: Intra-Oral LD Grinders (LD11M.RA or LD11C.RA) and Dialite Diamonds (Round, Football and Round-End Taper)

Power Suggestion: Brasseler Forza FI Electric Attachment with light pressure

Sterilization: Autoclave only

Recommended Speeds: 5,000-7,000 rpms

By Your Side In Dentistry



BRASSLER USA
DENTAL INSTRUMENTATION