solutions

Acrylic Temporization System







Figure Number	BR03	BR02	BR01
Head Size (½ mm)	220	220	220
Head Length (mm)	3.0	3.0	3.0







				-
Figure Number	H79EF	H257EF	365	305
Head Size (1/10 mm)	040	023	220	050
Head Length (mm)	13.0	13.0	_	_

Only Available Direct from your Brasseler Representative
To order call 800.841.4522 or fax 888.610.1937
In Canada call 800.363.3838 or fax 866.330.4454
Visit our website at BrasselerUSA.com

By

By Your Side In Dentistry



B-4022B-REV I-5023856L0-09.13-RRD

©2013 Brasseler USA. All rights reserved.

SOLUTIONS 711.11 Acrylic Temporization Kit

	Order #	Head Size	Grit	Usage		Order #	Head Size	Grit	Usage
0	BR01-HP	220	G	Polisher - Extra-Oral Acrylic Knife-Edge		H79EF-HP	040	3	Cutter - Extra-Fine for trimming and finishing alloys/acrylic
0	BR02-HP	Polisher - Extra-Oral Acrylic Knife-Edge		H257EF-HP	023	3	Cutter - Extra-Fine for trimming and finishing alloys/acrylic		
			Acrylic Knife-Edge	and the same of th					
0	BR03-HP	220	3	Polisher - Extra-Oral Acrylic Knife-Edge	0	365-HP	220	M	Diamond Disc - Serrated for Acrylics

TIPS & TECHNIQUES: Submerge provisional in alcohol for 60 seconds to remove the air inhibited layer prior to polishing to avoid the instruments loading up. Complimentary items: ET ProviPro (SWR22M & SWR22F)

Power Suggestion: Brasseler Forza L50K Laboratory Motor with light pressure

Sterilization: Autoclave only

Recommended Speeds: Polishers 8,000-10,000 rpms; Cutters 10,000-15,000 rpms

^{*}This kit is also ideal for adjusting and polishing all types of acrylic appliances including retainers, splints and night guards. By Your Side In Dentistry

