




ESX® TECHNIQUE TIPS

- Use all ESX files at 500-600 RPM with a torque setting of 1.5-2.3 Ncm.
- Always use lubrication with rotary files.
- Use a "LIGHT TOUCH" and the "SINGLE STROKE AND CLEAN" technique (after initial engagement, the file is removed from the canal and cleaned prior to re-engagement).

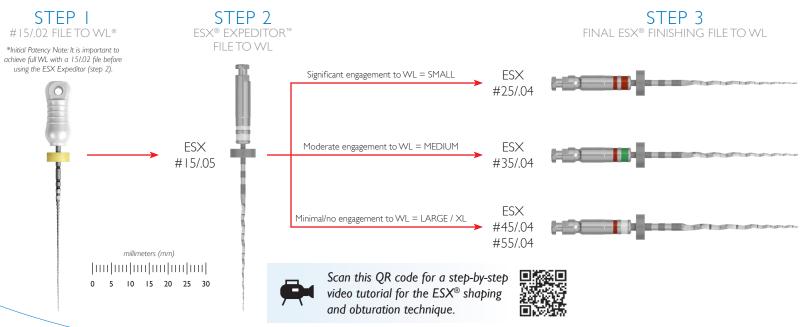


*For advanced cases (difficulty reaching WL with #15/.02 K-file) see reverse side.

To order call 800.841.4522 or fax 888.610.1937. Visit our website at BrasselerUSA.com







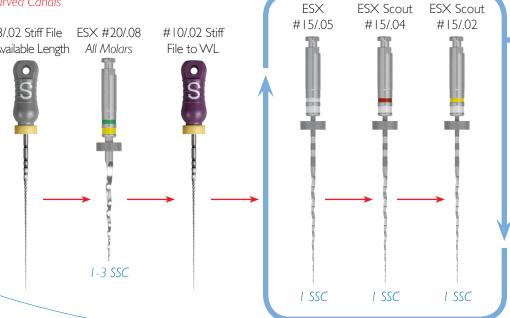

Repeat crown down cycle until #15/.02 WL is achieved.

Once #15/.02 WL is achieved, move to standard ESX sequence (#15.05 to WL and finish with appropriate ESX finishing file).

#15/.05 #25/.04

FSX

ESX



To order call 800.841.4522 or fax 888.610.1937. Visit our website at BrasselerUSA.com

