



Partial Extraction Therapy

A novel method for the socket shield technique as developed by Team Atlanta



K0297

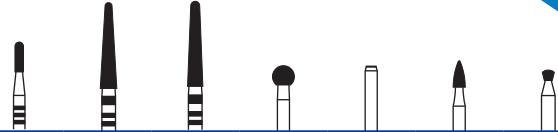


Figure No.	PET1	PET2	PET3	PET4	PET5	PET6	PET7
Head Size ($\frac{1}{10}$ mm)	012	018	018	029	016	016	014
Cutting Length (mm)	4.2	10	10	-	1.4	3.5	-
Grit	-	Medium	Medium	Coarse	Medium	Fine	Medium
Shank Type	FG Surg	FG Surg	FG Surg	FG Surg	FG Surg	FG Surg	FG Surg
Quantity	1	1	1	1	1	1	1

Only Available Direct from your Brasseler Representative

To order call 800.841.4522 or fax 888.610.1937

In Canada call 800.363.3838 or fax 866.330.4454

Visit our website at BrasselerUSA.com










B-4553B-502532110-2.16



Partial Extraction Therapy

A novel method for the socket shield technique as developed by Team Atlanta

Order #	Head Size	Grit	Usage
 PET1	012	-	Rapid removal of enamel and dentin
 PET2	018	M	Root Resection
 PET3	018	M	Root Resection
 PET4	023	C	Bone Shaping
 PET5	016	M	Bone Shaping - End Cutting
 PET6	016	F	Bone Shaping
 PET7	014	M	Bone Shaping

TIPS & TECHNIQUES: For optimal results, utilize the Zekrya gingival protecting kit

Complimentary items: Zekrya Gingival Protector, Zekrya large shields, Zekrya small shields, CR36/376 Rhodes Chisel, C3/46 Wedelstaedt Chisel and Periosteal Elevator

Handpiece Suggestion: Brasseler Forza F5

Sterilization: Autoclave only

Recommended Speeds: 160,000 rpm

Disclaimer:

Partial extraction therapy ("pet") is an advanced procedure used for the creation of a socket shield and should only be performed by licensed dental professionals with the requisite skill, education, and due care and who have received formal training on both this procedure and the placement of dental implants.

