**BRASSELER**  
USA®

∴BRiO440L

∴BRiO440

**HIGH SPEED AIR TURBINE HANDPIECE**

INSTRUCTIONS FOR USE

Read before use. Keep in a safe place for future reference.

**CAUTION:** Federal (US) law restricts this device to sale by or on the order of a licensed professional

Brasseler U.S.A. Dental, LLC hereinafter will be referred to as "Brasseler USA."

 **Caution**

- Do not run handpiece without a bur locked in chuck.
- Do not disassemble, modify or alter handpiece.
- Do not press push-button end cap during use since this could result in OVERHEATING of handpiece head. Special caution must be exercised during use to keep cheek tissue away from push-button end cap. Contact with cheek tissue may cause push-button to depress and may result in burn injury to patient.
- Never attempt to remove bur while handpiece is running.
- If any abnormalities are found, stop using immediately and contact Brasseler USA.

**Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.**

## SPECIFICATIONS

Description	Brio 440L	Brio 440
Item Code	5026058U0	5026059U0
Operating Air Pressure	38 - 42 psi	38 - 42 psi
Operating Speed	405,000 - 420,000 rpm	405,000 - 420,000 rpm
Connection Type	Brasseler fiber-optic	Brasseler non-optic
Power	20 W	20 W
Weight	50 g	50 g
Head Height	12.5 mm	12.5 mm
Head Diameter	10.5 mm	10.5 mm

### AIR PRESSURE

Set air pressure at handpiece connection to 38 - 42 psi.

**Caution**

- Exceeding recommended air pressure may result in premature cartridge failure.

## **OPERATION**

### **Coupler Connection (Fig. 1)**

1. Align tubes and pins on coupler with holes in delivery system tubing.
2. Thread tubing nut to bottom of coupler.
3. Hold coupler with provided wrench and finger tighten nut.

*Fig. 1*

## Handpiece Connection (Fig. 2)

### Attaching handpiece

1. Slide handpiece onto Brasseler coupler.
2. Press firmly until it snaps into place.

**Note:** Handpiece should swivel freely with slight tension and sit flush on coupler.

### Removing handpiece

Pull back the retention lock ring on coupler and pull handpiece straight off.

**Note:** Do not hold or pull on hose.

Fig. 2

## Changing Burs

### Caution

- Only use ISO standard burs with shank diameter of .0626" - .0630" (1.59-1.60 mm).
- Do not use bent, worn or damaged burs.
- Always keep burs clean. Dirt and debris on shank may cause clogging/sticking, loss of grip or premature wear of chuck.
- For standard head handpiece, use standard shank burs; to prevent bur breaking off in chuck assembly do not use short shank burs.

### Installation

1. Press push-button and insert bur into head until it stops (Fig. 3).
2. Release pressure on button; bur should now be secure.
3. Firmly tug on bur to ensure that it is properly secured.

### Removal

1. Press push-button on end cap with thumb (Fig. 3).
2. Pull to remove bur.

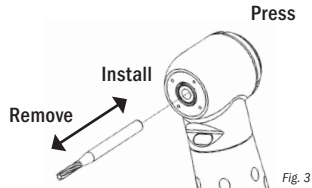
Fig. 3

## MAINTENANCE

**Caution**

- Wear appropriate personal protective equipment during maintenance process. This should include eye protection and gloves.

### Manual Cleaning and Lubrication

#### Handpiece Maintenance

**Note:** Cleaning and lubrication is required before each sterilization cycle.

1. Remove bur.
2. Detach handpiece from coupler.
3. Fit spray adapter onto universal cap nozzle of approved lubricant (Fig. 4).
4. Insert spray adapter completely into back of handpiece and hold it in place (Fig. 5).
5. Dispense spray for 1-2 seconds, then run handpiece to purge cleaner/lubricant.
6. Repeat steps 4 and 5 until cleaner/lubricant expelled from head is clear.
7. Wipe excess cleaner/lubricant from handpiece with dry, lint-free cloth.

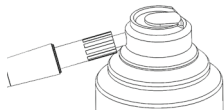
Fig. 4

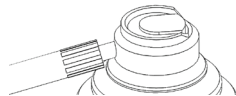
Fig. 5

## Chuck Maintenance

**Note:** It is recommended that the push-button chuck be cleaned every 3 - 4 maintenance cycles.

1. Remove bur.
2. Detach handpiece from coupler.
3. Fit chuck adapter onto universal cap nozzle of approved lubricant.
4. Insert spray adapter into chuck and dispense spray for 1 - 2 seconds (Fig. 6).
5. Work chuck mechanism to distribute cleaner/lubricant by repeatedly pressing button.
6. With push button depressed, insert and remove bur several times to break-up remaining debris.
7. Wipe off excess cleaner/lubricant with a dry, lint-free cloth.

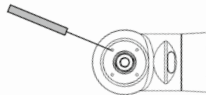
*Fig. 6*

## Cleaning Water Spray Ports

**Note:** It is recommended that spray ports be periodically cleaned.

1. Carefully clean ports using cleaning wire supplied with handpiece (Fig. 7).
2. Connect handpiece and run with water flow activated to confirm ports have been fully cleared of debris.
3. Repeat steps 1 and 2 as necessary.

*Fig. 7*

### **Cleaning Fiber-optic Rod**

- Clear fiber-optic surfaces of particulates by spraying with air syringe.
- Wipe fiber-optic surfaces with a soft cloth or cotton swab moist with alcohol.

### **STERILIZATION**

#### **⚠ WARNING**

- Sterilize by autoclave after each patient use.
- Sterilization is required before first use.

#### **⚠ Caution**

- Wear appropriate personal protective equipment during sterilization and lubrication process. This should include eye protection and gloves.
- Do not use ultrasonic bath or cold sterile/disinfectant chemicals to clean handpiece.
- Do not autoclave handpieces together with other instruments.
- Do not use dry heat sterilization.
- Keep handpiece in sterilization pouch until ready to use.
- Do not cool handpiece manually as damage may occur.

1. Remove bur, then wipe external surface with dry paper towel to remove any excess lubricant and debris. Place paper towel in bio-hazard bin when finished.
2. Place handpiece into sterilization pouch and seal. Bag handpieces individually; no more than 1 per pouch.
3. Steam autoclave for a minimum of 15 minutes at 275° F (135° C). Let autoclave cycle completely. Do not remove before handpiece is completely cool and pouch is dry.

### **REPLACEMENT PARTS & ACCESSORIES**

5010059U0	High speed Spray Nozzle
5021283U0	NL10LED Coupler for 6 Hole
5021281U0	NLT0TI Coupler for 6 Hole
5021282U0	NL11TI Coupler for 5 Hole
5021278U0	N10TI Coupler for 4 Hole

## **SERVICE**

For items that require service, including warranty service, please send to Brasseler USA.

Brasseler USA

Attn: Dental Repairs

One Brasseler Blvd.

Savannah, GA 31419

800.841.4522 (in Canada, call 800.363.3838)

## **WARRANTY**

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 1 year from date of purchase.

Brasseler USA will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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