

solutions™

Zirconia/PFM Crown Removal System



Order Number:
K0196



Figure Number	8801LDF	8856DF	1558SC	H34	H34L
Head Size ($\frac{1}{10}$ mm)	014	014	012	016	016
Head Length (mm)	—	8.0	4.0	2.0	3.5




Figure Number

EB134

Only Available Direct from your Brasseler Representative
To order call 800.841.4522 or fax 888.610.1937
In Canada call 800.363.3838 or fax 866.330.4454
Visit our website at BrasselerUSA.com

solutions™ K0196 Zirconia/PFM Crown Removal Kit

	Order #	Head Size	Grit	Usage
	H34-FG	012	N/A	Carbide - Metal Cutter
	H34L-FG	012	N/A	Carbide - Long Metal Cutter
	I558SC-FG	012	N/A	Carbide - Alloy Removal
	8801LDF-FG	014	F	Diamond - Occlusal Surface
	8856DF-FG	016	F	Diamond - Contact Surfaces/Crown Removal

TIPS & TECHNIQUES: Dialite Diamonds are recommended to cut through ceramics only and metal cutters are recommended for alloy removal.

Power Suggestion: Forza F5 electric attachment.

Sterilization: Autoclave

Recommended Speeds: Diamonds 120,000 rpms; Carbides 300,000 rpms

Order #	Usage
EB134	Hand Instrument - Crown Spreader



By Your Side In Dentistry



BRASSLER USA
DENTAL INSTRUMENTATION