

solutions™

ET Illustra™ Introductory System



Order Number:

K0113



Figure Number	132RA	232RA	138RA	238RA	135RA	235RA
Head Size ($\frac{1}{10}$ mm)	050	050	100	100	060	060
Head Length (mm)	10.0	10.0	1.0	1.0	8.0	8.0




Only Available Direct from your Brasseler Representative




To order call 800.841.4522 or fax 888.610.1937

In Canada call 800.363.3838 or fax 866.330.4454

Visit our website at BrasselerUSA.com

solutions™ K0113 ET Illustra™ Introductory System

	Order #	Head Size	Grit	Usage
	I32RA-RA	050	M	Polisher - Occlusal Surfaces
	I35RA-RA	060	M	Polisher - Labial/Lingual/Buccal
	I38RA-RA	100	M	Polisher - Interproximal Surfaces

	Order #	Head Size	Grit	Usage
	232RA-RA	050	F	Polisher - Occlusal Surfaces
	235RA-RA	060	F	Polisher - Labial/Lingual/Buccal
	235RA-RA	100	F	Polisher - Interproximal Surfaces

TIPS & TECHNIQUES: Polishers produce best results when used in order (medium and fine); works well with harder nano composites.

Complimentary items: ET Finishing Carbides and Diamonds, ET Flex Interproximal Strips

Power Suggestion: Use Brasseler Forza FI Electric Attachment with light pressure

Sterilization: Autoclave or Single Use

Recommended Speeds: Dark Purple Medium - 10,000-12,000 rpms; Light Purple Fine - 2,000-10,000 rpms

By Your Side In Dentistry



BRASSLER USA
DENTAL INSTRUMENTATION