

# solutions™

## Dialite® ZR Extra-Oral Polishing System for Zirconia



Order Number:  
**K0238**



Figure Number	L20MZR.HP	L20FZR.HP	H8MZR.HP	H8FZR.HP
Head Size ( $\frac{1}{10}$ mm)	200	200	110	110
Head Length (mm)	1.0	1.0	2.0	2.0



Figure Number	R17MZR.HP	R17FZR.HP	H2MZR.HP	H2FZR.HP	LD15C
Head Size ( $\frac{1}{10}$ mm)	170	170	040	040	150
Head Length (mm)	2.5	2.5	13.0	13.0	3.0




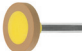
Only Available Direct from your Brasseler Representative  
To order call 800.841.4522 or fax 888.610.1937  
In Canada call 800.363.3838 or fax 866.330.4454  
Visit our website at [BrasselerUSA.com](http://BrasselerUSA.com)


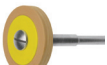



By Your Side In Dentistry



B-3793B-REVI-5022152L0-01.13-RRD

# solutions™ K0238 Dialite® ZR Extra-Oral Polishing System for Zirconia

	Order #	Head Size	Grit	Usage
	L20MZR.HP	200	M	Polisher - Interproximal Surfaces
	L20FZR.HP	200	F	Polisher - Interproximal Surfaces
	H8MZR.HP	110	M	Polisher - Contact Surfaces
	H8FZR.HP	110	F	Polisher - Contact Surfaces

	Order #	Head Size	Grit	Usage
	R17MZR.HP	170	M	Polisher - Contact Surfaces
	R17FZR.HP	170	F	Polisher - Contact Surfaces
	H2MZR.HP	040	M	Polisher - Occlusal Surfaces
	H2FZR.HP	040	F	Polisher - Occlusal Surfaces
	LD15C.HP	150	C	Grinder - Adjusting

**TIPS & TECHNIQUES:** Polishers produce best results when using both grits in order (medium and fine); medium green for pre-polishing and fine orange for final polishing.

Any adjustments should be made with the LD Grinder in coarse Green.

**Complimentary items:** Dialite Extra-Oral Diamond Paste (5015241U0), Diamond Discs (911H, 911HH, 911HV)

**Power Suggestion:** Brasseler Forza L50K Laboratory Motor with light pressure

**Sterilization:** Autoclave only

**Recommended Speeds:** 7,000-10,000 rpms

By Your Side In Dentistry



**BRASSLER USA**  
DENTAL INSTRUMENTATION