




BRASSLER
USA®

KontrolFlex™

Rotary Files


Controlled Efficiency

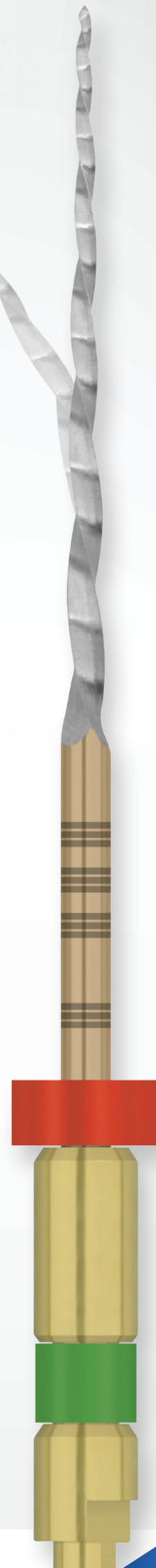

BY YOUR SIDE

THE RIGHT INSTRUMENT FOR THE JOB

Brasseler USA's exclusive KontrolFlex files strike the ideal balance between cutting efficiency and safety. Unlike traditional NiTi files, KontrolFlex files feature Controlled Memory NiTi Technology. This patented technology alters the properties of NiTi allowing the files to be pre-curved and highly resistant to cyclic fatigue. Other "Controlled Memory" files on the market can experience a drastic reduction in cutting efficiency as a result of the softened metal. The KontrolFlex file was developed to meet the needs of endodontic specialists who encounter extremely difficult curved and calcified cases.

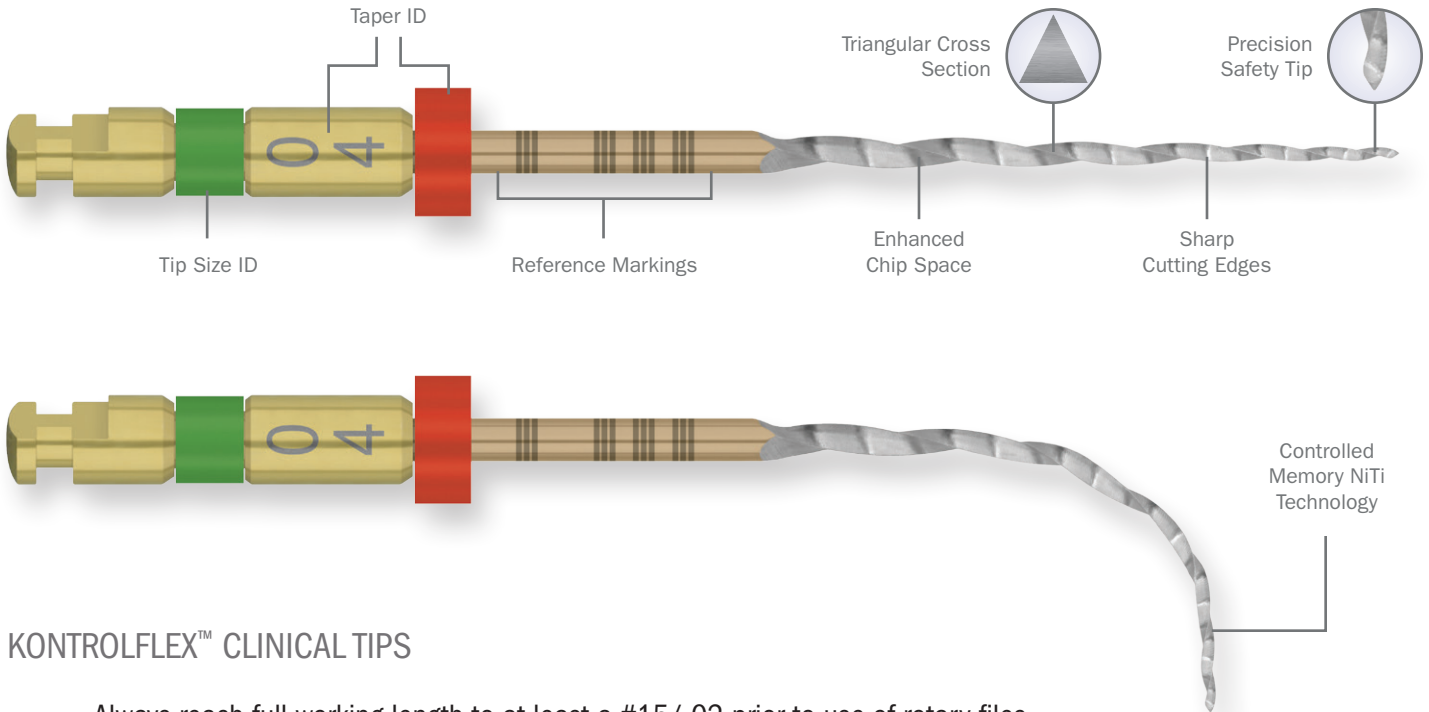

KONTROLFLEX™ CLINICAL TIPS

- Always reach full working length to at least a #15/.02 prior to use of rotary files.
- 500-600 RPMs.
- 1.8-2.3 Ncm.
- Use a feather-light touch taking the file to engagement (not resistance).
- Use the Single Stroke and Clean Technique. Clean the flutes after each gentle engagement.
- Irrigate between each rotary file.








KONTROLFLEX™ PRODUCT OFFERING (4/PACK)

File/ Tip ID	Tip Size	.04 Taper (Stopper)		.05 Taper (Stopper)		.06 Taper (Stopper)		.07 Taper (Stopper)
		21 mm	25 mm	21 mm	25 mm	21 mm	25 mm	19 mm
△ White	15	5024990U0	5024996U0	-	-	-	-	-
▲ Purple	17	-	-	5025065U0	5025066U0	-	-	-
▲ Yellow	19	-	-	-	-	-	-	5025064U0
▲ Yellow	20	5024991U0	5024997U0	-	-	5025067U0	5025068U0	-
▲ Red	25	5024992U0	5024998U0	5025069U0	5025070U0	-	-	-
▲ Blue	30	5024993U0	5024999U0	5025071U0	5025072U0	-	-	-
▲ Green	35	5024994U0	5025000U0	-	-	-	-	-
▲ Black	40	5024995U0	5025001U0	-	-	-	-	-
△ White	45	5025009U0	5025010U0	-	-	-	-	-



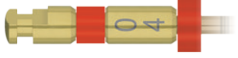





OUR MOST FLEXIBLE FILE

KontrolFlex files can be utilized as a stand-alone shaping system or as adjunct files for specific clinical situations (difficult access, extreme curvatures, etc). The extensive range of tip sizes and tapers allows you to quickly incorporate the KontrolFlex files that best meet your treatment philosophy.

CONSTANT TAPER SHAPING PHILOSOPHY

KontrolFlex™	ProFile® Vortex®/ Vortex Blue™	EndoSequence®	X7
	15/.04	15/.04	15/.04
	20/.04	20/.04	20/.04
	25/.04	25/.04	25/.04
	30/.04	30/.04	30/.04
	35/.04	35/.04	35/.04
	40/.04	40/.04	40/.04
	45/.04	45/.04	45/.04

VARIABLE TAPER SHAPING PHILOSOPHY


KontrolFlex™	ProTaper®/ ProTaper Gold™	X3
	SX	NX
	S1	N1
	S2	N2
	F1	C1
	F2	C2
	F3	C3
	N/A	N/A
	F4	C4



DIAMONDS



CARBIDES

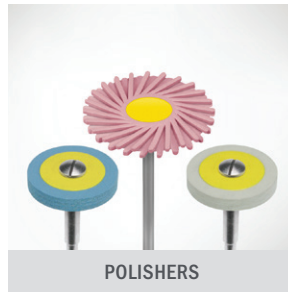

POLISHERS



PROCEDURE SYSTEMS



BIOCERAMICS



Since 1976, our absolute focus has been to develop products and provide services that support the practice of core operatory procedures. Today, Brasseler USA® is the premier dental instrumentation company in North America offering the most extensive selection of dental and surgical instrumentation under one brand.



HANDPIECES



ENDODONTICS



SMALL EQUIPMENT



HYGIENE



HAND INSTRUMENTS



ORAL SURGERY



LABORATORY INSTRUMENTS



Visit our website at BrasselerUSA.com

To order call 800.841.4522 or fax 888.610.1937. In Canada call 800.363.3838

