



BRASSELER
USA®

BY YOUR SIDE

Gold Grip Esthetic Contouring Instrument

► The Challenge

Manipulation of composite can be tedious and often results in an irregular surface. With traditional instruments, it often becomes necessary to go over the composite many times in an attempt to smooth the material. Highly detailed anterior composite restorations require specifically designed instruments to properly manipulate and contour the material, especially along the cervical margins.

► The Solution

The Gold Grip Esthetic Contouring Instrument makes anterior composite restoration easy! Features a convex-shaped paddle that allows you to spread composite across the facial plane for diastemas, Class IV, and Class V restorations. The convex surface has no edge to catch and pull the composite, allowing for a smooth composite-to-tooth transition.



Clinician's Choice

► Order online at BrasselerUSA.com or call **800.841.4522**.



©2021 Brasseler USA. All rights reserved. Printed in the U.S.A.

Long, flat, thin,
Flexible Blade
makes interproximal
composite placement
and manipulation
effortless



Convex Paddle
allows for easy and
smooth spreading
of composite across
the facial plane

B-5542-08.21

Gold Grip Esthetic Contouring Instrument

► Clinical Technique



The convex paddle of the Gold Grip Esthetic Contouring Instrument is a sturdy, curved spoon shape, the back of which is ideal for shaping the curves of lingual concavities. The front side is used to form the curvature at the gum line or interproximal surfaces on the lingual side and may also be used to carry restorative composite material, such as Evanesce™, to the preparation.



The long, flat, thin, flexible blade of the Gold Grip Esthetic Contouring Instrument is spatula-shaped and designed for use on facial surfaces of maxillary incisors, buccal surfaces of posterior teeth, and where there is a large, rounded surface. It is excellent for adapting a gum line margin and ideal for forming a gentle curve toward interproximal spaces and the tooth's edge for esthetic highlighting.

► Product Offering



Gold Grip Esthetic Contouring Instrument

Order No. 063103

► Order online at [BrasselerUSA.com](https://www.BrasselerUSA.com) or call **800.841.4522**.

