



**BRASSELER**  
USA®

BY YOUR SIDE

# Composite Ninja

## Comfort Grip

*Maximize* your Esthetic Composite Potential.  
*Minimize* your Chairtime and Instrumentation.

### ► The Challenge

One of the challenges of delivering a high-quality, esthetic composite restoration is choosing and using the appropriate instrumentation. The stages of placement, shaping and contouring, and finishing require specific instrumentation for each specific function.

### ► The Solution

Less instrumentation means less inventory, less set-up, and less to sterilize. The Composite Ninja Comfort Grip, designed by Dr. Les Kalman, performs the tasks of several composite instruments, replacing the idea of a single instrument for a single purpose. Its scope of usefulness is only limited by the imagination.




**Clinician's Choice**

► Order online at [BrasselerUSA.com](https://www.BrasselerUSA.com) or call **800.841.4522**.



# Composite Ninja Comfort Grip

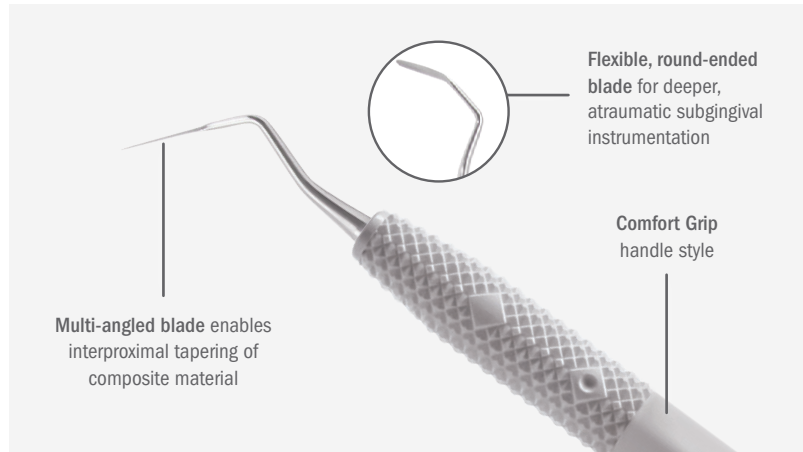

## Versatile for Placement, Shaping, and Carving

Featuring an angulated, sharp, two-sided blade, the Composite Ninja Comfort Grip allows the clinician to easily access any area of the tooth surface without an awkward rotation of the wrist. This unique flexible blade is longer than a traditional periodontal scaler and round-ended for deeper, atraumatic subgingival instrumentation.

## Saves Finishing Time

Saving finishing time is essential to efficiency and profitably providing quality restorations. The Composite Ninja Comfort Grip allows for adapting and shaping the composite before curing. Its multi-angled blade enables the tapering of the composite interproximally while establishing line-angles and blending the material into the natural tooth, forming a restoration that requires minimal post-curing adjustment.



*Clinical photography courtesy of Dr. Bob Margeas*

## Gently Removes Flash

Even with utmost care and attention in placing the composite, it is rare to have a restoration free of composite or adhesive flash after curing. The use of a handpiece in these cases may be too aggressive and traumatic to the gingiva, and may alter surface anatomy. The Composite Ninja Comfort Grip allows sensitive tactile feedback to the dentist while its long, sharp blade accurately and efficiently removes any unwanted material.

## ► Product Offering



Composite Ninja Comfort Grip

Order No.

060105

► Order online at [BrasselerUSA.com](https://www.BrasselerUSA.com) or call **800.841.4522**.

