

Disposable Aluminum Temporary Crown and Bridge Impression Trays - Posterior & Anterior

INSTRUCTIONS FOR USE

DATE OF ISSUE: November 1, 2022

INDICATIONS FOR USE:

Disposable aluminum tray for the fabrication of a temporary matrix.

INSTRUCTIONS FOR USE:

1. Load the tray with a matrix material (Accur8™ or Matrix1™) or other suitable impression material, following manufacturer's instructions for use.
2. Seat filled tray intraorally and allow the material to cure.
3. Remove the tray and proceed with provisional fabrication.

NOTE: Trays are not sold in sterile condition.

STORAGE:

Trays should be kept in its original packaging at room temperature and protected against dust, direct sunlight, moisture and corrosive gas until use.

SYMBOLS USED:



Consult instructions for use or consult electronic instructions for use



Do Not Reuse



Caution



Non Sterile



Keep Dry



Keep Away From Sunlight

Rx Only Prescription Only



Distributor



Quantity



Catalog Number



Batch Code



800.841.4522
BrasselerUSA.com



Brasseler U.S.A. Dental, LLC
One Brasseler Blvd
Savannah, GA 31419, U.S.A.