BRASSELER
USA®

HygienePro
AIR 2

INSTRUCTIONS FOR USE

Read before use. Keep in a safe place for future reference.

CAUTION: Federal (US) law restricts this device to sale by or on the order of a licensed professional

Brasseler U.S.A. Dental, LLC hereinafter will be referred to as “Brasseler USA.”

Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
 WARNING	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed

Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.

Symbols	
SN	Serial Number
REF	Reference Number

QTY	Quantity
	Instructions for Use

Caution

- For professional use only.
- The user is responsible for correct operation, maintenance, and inspection.
- Sterilization is required before first use.
- Do not wipe or immerse in acidic solutions.
- Do not modify or alter coupler.
- If handpiece leaks air or water and/or does not run up to speed, check o-rings or seals on coupler.

- Never attempt to remove the coupler while running handpiece.
- If any abnormalities are found, stop using immediately and contact Brasseler USA.

SPECIFICATIONS

Item Code	Description	Weight
5030744U0	HygienePro Air 2 SW Coupler	5g

AIR PRESSURE

Set air pressure at handpiece connection to 36 psi.

Caution

- Exceeding recommended air pressure may result in premature failure.

OPERATION

Connecting Coupler (Fig. 1)

1. Align tubes and pins on coupler with holes in delivery

system tubing.

2. Thread tubing nut to bottom of coupler.
3. Hold coupler and finger tighten nut.

Fig. 1

Connecting Handpiece (Fig. 2)

1. Slide handpiece onto coupler.
2. Press firmly until it snaps into place.

Note: Handpiece should swivel freely with slight tension and sit flush on coupler.

Removing Handpiece (Fig. 2)

1. Hold ring on coupler and pull handpiece straight off.

Note: Do not hold or pull on hose.

Fig. 2

MAINTENANCE

NOTE: The HygienePro Air 2 & 2 SW **DO NOT** require lubrication. If the handpiece becomes sluggish, use the following procedure.

1. Disconnect handpiece and coupler from air supply tubing. Leave the coupler in the handpiece.
2. Dispense handpiece cleaner into drive air tube of handpiece or coupler (Fig. 3).
3. Reconnect handpiece and run for 10 seconds or until handpiece runs smoothly.

Fig. 3

Changing Coupler O-rings

- Improper care of o-rings may cause handpiece to malfunction.
- When changing/replacing coupler o-rings, always replace complete set of 2 pieces.

Note: Coupler o-rings may be lubricated with Brasseler Handpiece Spray.

1. Use an explorer to gently remove old/worn o-rings.

2. Slide on new o-rings using only fingers to not cause damage.

REPLACEMENT PARTS & ACCESSORIES

5029861U0 Brasseler Handpiece Spray

5030745U0 HygienePro Air 2 SW Coupler O-Ring Kit

SERVICE

For items that require service, including warranty service, please send to Brasseler USA.

Brasseler USA

Attn: Dental Repairs

One Brasseler Blvd.
Savannah, GA 31419

800.841.4522 (in Canada, call 800.363.3838)

WARRANTY

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 2 years from date of purchase.

Brasseler USA will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any

other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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