**BRASSELER**  
USA®

*HygienePro* *HygienePro*  
AIR 2                                    AIR 2 SW

INSTRUCTIONS FOR USE

Read before use. Keep in a safe place for future reference.

**CAUTION:** Federal (US) law restricts this device to sale by or on the order of a licensed professional

Brasseler U.S.A. Dental, LLC hereinafter will be referred to as "Brasseler USA."

Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
 <b>WARNING</b>	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed
 <b>Caution</b>	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.

Symbols	
	Serial Number
	Reference Number
	Quantity
	Instructions for Use

### Caution

- For professional use only.
- The user is responsible for correct operation, maintenance, and inspection.
- Sterilization is required before first use.
- Do not wipe or immerse in acidic solutions.
- This handpiece uses only disposable prophy angles.
- Never operate the handpiece without a disposable prophy angle installed.
- Before each use, check for vibration, noise, and overheating. If any abnormalities are observed, stop using immediately and contact Brasseler USA.
- Do not exceed the recommended air pressure. Excessive air pressure may cause premature failure.
- Do not drop, hit or subject to excessive shock.
- Do not disassemble, modify or alter handpiece.
- Never attempt to remove disposable prophy angle while handpiece is running.

## Specifications

Description	HygienePro Air 2	HygienePro Air 2 SW
Item Code	5030742U0	5030743U0
Drive Air Pressure	36 psi	36 psi
Operating Speed	0 - 3,000 rpm	0 - 3,000 rpm
Connection Type	4-hole	Swivel Coupler
Weight	89 g	86 g

## Operation

### Caution

- Air supply must be dry, clean, and uncontaminated. Moist, contaminated air leads to wear and premature failure.
- Exceeding recommended air pressure may result in premature failure.
- Damaged or missing connection gasket can cause water and air leaks resulting in poor performance.
- Improper connection to tubing may result in handpiece disconnecting while in use. Before each use, ensure handpiece is securely fastened to tubing.

### HYGIENEPRO AIR 2

#### Connecting Handpiece (Fig. 1)

1. Align tubes in handpiece with delivery system tubing.
2. Thread tubing nut to bottom of handpiece and finger tighten.

#### Removing Handpiece (Fig. 1)

1. Unthread tubing nut and pull handpiece straight off.

## HYGIENEPRO AIR 2 SW

### Connecting Coupler (Fig. 2)

1. Align tubes and pins on coupler with holes in delivery system tubing.
2. Thread tubing nut to bottom of coupler.
3. Hold coupler and finger tighten nut.

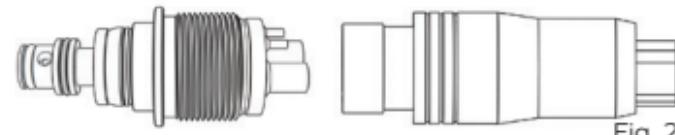
Fig. 2

### Connecting Handpiece (Fig. 3)

1. Slide handpiece onto coupler.
2. Press firmly until it snaps into place.

**Note:** Handpiece should swivel freely with slight tension and sit flush on coupler.

Fig. 3

### Removing Handpiece (Fig. 3)

1. Hold ring on coupler and pull handpiece straight off.

**Note:** Do not hold or pull on hose.

### Changing Color Bands

**Note:** Color bands are autoclavable.

1. Use an explorer to gently remove old color band.
2. Slide on new color band using only fingers to not cause damage.

## CHANGING DISPOSABLE PROPHY ANGLE (FIG. 4)

### Caution

- Make sure the prophy angle sits flush on handpiece. If not, repeat inserting process.
- Do not install or remove the prophy angle until motor has stopped completely.
- Loose prophy angles cause excessive noise and vibration and may result in patient injury.
- After the prophy angle is seated, pull on it slightly to make sure it is locked into place.
- Do not re-use disposable prophy angles as they could break and cause injury.

### INSTALLATION

1. Align notch on prophy angle with alignment pin on hand-piece and push the prophy angle on until it stops.

### REMOVAL

1. To remove prophy angle, pull off by hand.

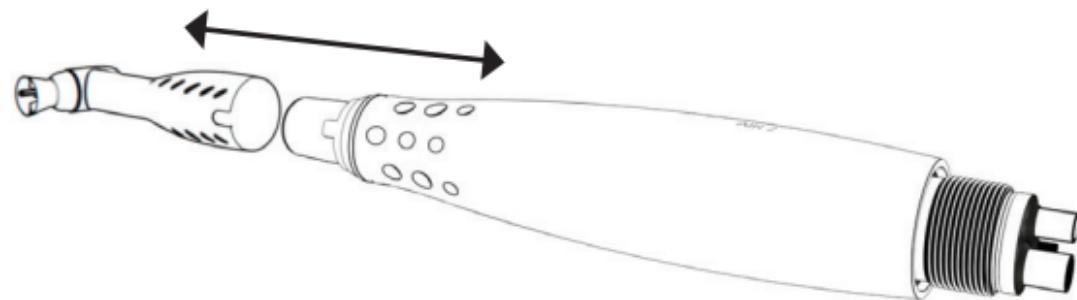
Fig. 4

## Maintenance

**NOTE: The HygienePro Air 2 & 2 SW DO NOT require lubrication. If the handpiece becomes sluggish, use the following procedure.**

1. Disconnect handpiece and coupler from air supply tubing. Leave the coupler in the handpiece.
2. Dispense handpiece cleaner into drive air tube of handpiece or coupler (Fig. 5).
3. Reconnect handpiece and run for 10 seconds or until handpiece runs smoothly.

Fig. 5

### CHANGING COUPLER O-RINGS

- Improper care of o-rings may cause handpiece to malfunction.
- When changing/replacing coupler o-rings, always replace complete set of 2 pieces.

**Note:** Coupler o-rings may be lubricated with Brasseler Handpiece Spray.

1. Use an explorer to gently remove old/worn o-rings.
2. Slide on new o-rings using only fingers to not cause damage.

## Sterilization

### Warning

- Sterilize with autoclave after each patient use.
- Sterilization is required before first use.

### Caution

- Wear appropriate personal protective equipment during sterilization and lubrication process. This should include eye protection and gloves.
- Do not use ultrasonic bath or cold sterile/disinfectant chemicals to clean handpiece.
- Do not autoclave handpieces together with other instruments.
- Do not use dry heat sterilization.
- Keep handpiece in sterilization pouch until ready to use.
- Do not cool handpiece manually as damage may occur.
- Do not autoclave at temperatures that exceed 275° F (135° C).

1. Remove bur, then wipe external surface with dry paper towel to remove any excess lubricant and debris. Place paper towel in bio-hazard bin when finished.
2. Place handpiece in sterilization pouch and seal. Bag handpieces individually; no more than 1 per pouch.

3. Sterilize in a gravity type autoclave for a minimum of 15 minutes or in a pre-vacuum autoclave for a minimum of 4 minutes at 270° F (132° C). Let autoclave cycle completely. Do not remove before handpiece is completely cool and pouch is dry.

## Replacement Parts & Accessories

5030744U0	HygienePro Air 2 SW Coupler	5030745U0	HygienePro Air 2 SW Coupler O-Ring Kit
5030746U0	HygienePro Air 2 Bands - Pink	5030747U0	HygienePro Air 2 Bands - Red
5030748U0	HygienePro Air 2 Bands - Orange	5030749U0	HygienePro Air 2 Bands - Blue
5030750U0	HygienePro Air 2 Bands - Green	5029861U0	Brasseler Handpiece Spray

## Service

For items that require service, including warranty service, please send to Brasseler USA.

Brasseler USA

Attn: Dental Repairs

One Brasseler Blvd.

Savannah, GA 31419

800.841.4522 (in Canada, call 800.363.3838)

## **Warranty**

These products are designed for use in a dental office and this warranty is not applicable to other uses. The products are warranted against manufacturing defects in materials and workmanship for 2 years from date of purchase.

Brasseler USA will repair or replace products at its sole discretion. This warranty shall be limited to replacement or repair of the products or their parts and shall not extend to any other claims, including but not limited to loss of profit, cost of removal or replacement, incidental or consequential damages or other similar claims arising from use of this product. Damage to products resulting from acts of God, faulty installation, misuse, tampering, accident, abuse, negligence, alterations, unauthorized repairs or problems unrelated to materials and workmanship are not covered by this warranty.

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