



**BRASSELER**  
USA®



# EDGEPRO™

## Techniques

BY YOUR SIDE

# Solutions Techniques

## ► EdgePRO® Mid-Root with Solutions Technique

99% Cleaning and Disinfection has been shown by only taking the EdgePRO® Tip #2 to Mid-Root with Irrigation Solutions.

By using EdgePRO® with the most Advanced Laser Light Sound Technology with the larger EdgePRO® Tip #2, you do not have to take the tip further down the canal than Mid-Root to get 99% Cleaning and Disinfection. The larger EdgePRO® Tip #2 Mid-Root extends the Photo Acoustic Wave which allows for the Sound Cleaning, Light Speed Cleaning and Disinfection effect of EdgePRO® throughout the Root Canal System.

Yes by only taking EdgePRO® Tip #2 to Mid-Root, it is Cleaning and Disinfecting the Root Canal System Working Length, Apical 1/3, Middle 1/3, Coronal 1/3, Lateral Canals, Accessory Canals...If it's the Root Canal System, it's Cleaning and Disinfecting!

We have found for clinicians new to our amazing EdgePRO® Cleaning and Disinfecting Technology, this is a great introduction option to EdgePRO®.

And we find most like to stay with this great option for its high cleaning and disinfection effectiveness and it's ease of use.

## ► EdgePRO® Working Length Minus 1mm with Solutions Technique

While we like to introduce Clinicians to EdgePRO® with Solutions using the larger Tip #2 going to Mid-Root, some are going to want to eventually go to the Working Length—Minus 1mm.

Amazing results of 99% Cleaning and Disinfection have been shown by taking the smaller EdgePRO® Tip #1 to 1mm Short of the Working Length with Irrigation Solutions.

By using EdgePRO® with the most Advanced Laser Light Sound Technology with the smaller EdgePRO® Tip #1, you can go where you want to go... and the EdgePRO® #1 Tip will take you there—Working Length Minus 1mm for 99% Cleaning and Disinfection.

Why Working Length Minus 1mm?

The Laser Light Sound Energy extends 1mm from the tip and while it's not a problem, it's best to keep EdgePRO® Tip #1, 1mm from the Working Length so it does not dampen the Laser Light Sound Energy which creates the Photo Acoustic Wave that Cleans and Disinfects throughout the Root Canal System. That's why it's better if you give EdgePRO® Tip #1 Working Length Minus 1mm a little space to work.

So—You get Great Clean and You get to be You—Working Length—Minus 1mm

# Distilled Water Technique

## ► EdgePRO® Distilled Water Technique

For those Clinicians that have been wanting to move their endodontic treatment away from chemical based solutions like NaOCl, EDTA, Chlorhexidine or Antibiotic containing solutions to a more gentler and natural cleaning and disinfection technique but not compromise cleaning and disinfection!

Now with the EdgePRO® Distilled Water Technique— You Don't!

By using the most Advanced Laser Light Sound Technology— EdgePRO® delivers up to 99% cleaning and disinfection all by using distilled water.

All the cleaning and disinfection you demand... without the solutions!