Instructions for Use

Dental Root Canal Instruments

Product Name: KontrolFlex™ Standard K-Files
KontrolFlex™ Stiff K-Files
KontrolFlex™ Hedstrom K-Files

Type/model: K, H

Specifications: 06, 08, 10, 15, 20, 25, 30, 35, 40, 45, 50, 55

Intended Use

The product is a dental repair instrument for the removal of dentin and root canal forming, the treatment of various pulpitis, pulp necrosis of root canal disease.

Structure and Composition

The hand root canal file consists of working part, handle and stopper.

Main Materials

Working part: Stainless Steel or NiTi Alloy

Handle: Plastic PBT

Stopper: Silicone Rubber

Contraindication

The product contains Nickel and chromium and should not be used for individuals with known allergic sensitivity to this metal.

Warnings

1. This product can only be used by professional dentists.
2. Ensure that the product is disinfected every time it is used.
3. Do not use this product for any purpose other than dental treatment and service.
4. It is highly recommended to use a rubber dam system during the endodontic procedure.

Precautions for Use

1. For your own safety, please wear personal protective equipment (gloves, glasses, mask).
2. To prevent infection, please disinfect the product with autoclave before use.
3. This product should be treated as medical waste.

Directions for Use

1. Open the cavity of crown, using diamond burs.
2. Explore the root canal with a #8 or #10 file, to follow the root canal by rotating in a back-and-forth motion. Use sodium hypochlorite to flush root canal.
3. Use a #15 file to further smooth the root canal and then determine the working length.
4. Flush again and use a #15 file to remove the debris to avoid obstruction.
5. Use #20 file, to pull up and down. Each time prepare 2-3MM, then clean debris and prepare gradually to the apex.
6. Use #25 file to repair the root canal. If resistance is encountered during preparation, do not force the file apically. Use sodium hypochlorite to wash the root canal throughout the procedure to prevent debris from being backed toward the apex, resulting in a filling failure.
7. For large root canals, select the appropriate size files and prepare the canal sequentially to the desired master apical size for obturation.

Cleaning, Disinfection and Sterilization

1. After unpacking, the root canal file must be sterilized according to the requirements before use. The recommended sterilization method is high temperature and high pressure steam sterilization at 134°C for 18min.
2. After use, wash with running water and clean the residual debris on the file with a soft brush.
3. Then, soak the product in a solution containing proteolytic enzyme for disinfection.
4. The product should be re-sterilized before use, the recommended sterilization method is 134°C for 18min.

Product Storage Conditions

1. Handle with care during transportation and storage; do not stack to avoid damage.
2. Avoid storing in areas with high temperature, high humidity or direct sunlight. The product should be stored at room temperature.

Glossary of Symbols: www.Brasselerusadental.com/resources

Manufactured for
Brasseler U.S.A. Dental, LLC
One Brasseler Blvd.
Savannah, GA 31419 USA
Telephone 800-841-4522

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